


DECOLONIZATION OF INFECTED WOUND WITH PRINCIPELLE IF IMPREGNATED WOUND DRESSING.

t=2 days



26-7

Male (age 75). Infected wound (30 days old). Antibacterial treatment and autolytic debridement initiated.

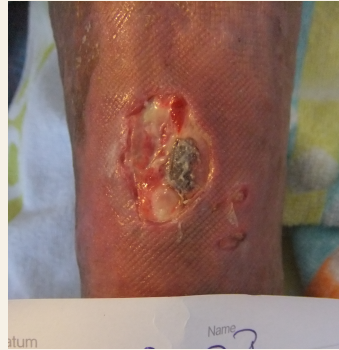

12-8

Continuation of antibiotic treatment; dissolve necrotic tissue.

Dressing changed every 2 days during treatment.



04-9

Necrotic tissue drained, allowing more space for granulation tissue.

Goal: stabilising the wound and removing yellow debris.

t=48 days



11-9

Goal of treatment (decolonization and cleaning of the wound) reached. Periwound skin irritation decreased, surface of the wound decreased.

The wound is treated further with a PU-foam wound dressing.

Product used:

Principelle IF (PPIF081010)
Sterile impregnated wound dressing.

