

# 1<sup>ST</sup> AND 2<sup>ND</sup> DEGREE BURN TREATED WITH PRINCIPELLE IF AND PRINCIPELLE MATRIX

t=0 days



17-02

Female (age 35) presented 1<sup>st</sup> and 2<sup>nd</sup> degree non-chemical burn.

Blisters were punctured to release tension. Wound was treated with Principelle IF ointment and Principelle Matrix.



22-03

Dressings changed every 7 days. Principelle Matrix dressing was removed without pain.


## Product used:

Principelle IF Honey-based Antibacterial ointment (PPIF15)



## Product used:

Principelle Matrix (PPM1010) Sterile impregnated wound dressing

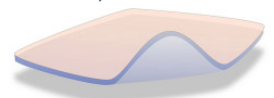

t=19 days



08-03

Wound fully healed within 3 weeks without loss of function, side effects or scarring.