Principelle MATRIX[®]

Self-adhesive super-absorbent dressing with integral film backing, pre-moistened with 30% water.

low-moderate exudate

UK - Principelle Matrix[®]

Instructions for use

PRODUCT DESCRIPTION

Principelle Matrix is a patented copolymer gel dressing with an integral breathable PU film barrier and a wound-facing matrix layer. Two versions are available:

Principelle Matrix
Principelle Matrix Border

Principelle Matrix Border has an additional skin friendly, adhesive border (PU film barrier) causing the dressing to be more occlusive.

Both versions of Principelle Matrix have been formulated and designed specifically to encourage wound bed preparation, granulation and subsequent epithelialisation of chronic wounds, while minimizing pain levels and the risk of infection. The dressings help to provide an environment for optimal wound healing by managing wound exudate levels and protecting against wound dehydration and external bacterial contamination. The super-absorbent gel provides both cushioning and absorption. The partially-hydrated nature of the dressing aids the process of autolytic debridement and helps soothe the wound on contact.

INDICATIONS

Under the supervision of a health care professional Principelle Matrix may be used for the management of acute and chronic, non-infected or mildly infected wounds. It may be used on moderately exuding, lightly exuding and dry wounds. The dressing can also be used on more highly exuding wounds, though this will reduce the effective wear time of the dressing and the wound should be monitored accordingly.

CONTRA-INDICATIONS

Principelle Matrix is not indicated for full-thickness wounds, heavily bleeding wounds, third degree burns, or as a cover for deep narrow cavities or sinuses.

METHOD OF USE

Principelle Matrix (borderless) can be placed directly onto the surface of the wound. It will be held in place by its adhesive nature, but a secondary dressing may also be necessary for long-term wear of the borderless product or if the dressing is to be exposed to external moisture (showering or bathing).

Principelle Matrix Border with the skin friendly, adhesive border (PU film layer) is available if extra fixation is required.

Principelle Matrix Border is also indicated for long-term wear, or if the dressing is to be exposed to external moisture (showering or bathing).

DIRECTIONS FOR USE

- Allow Matrix to completely cover the wound and extend onto healthy tissue by approximately 15 to 20mm depending on the level of exudate.
- Prepare the wound site by cleansing as needed; dry the surrounding skin.
- Remove the sterile dressing from the package.
- To apply the dressing, first remove part of the overlaid white plastic liner.
- Position and smooth into place whilst removing the second half of the white plastic liner.
- For Principelle Matrix Border:** once the dressing is securely in place, peel away the supporting layer from the PU film.
- For a borderless dressing:** you may apply a secondary film dressing or bandage as required. To encourage autolytic debridement (by reducing vapour loss from the dressing), place an adhesive film over the entire dressing.
- Check the wound regularly. The dressing should be changed as often as the condition of the wound dictates, but as a general rule, the dressing should be replaced after 3 – 7 days. In the treatment of a highly exuding wound, it will be necessary to change the dressing more frequently.

PRECAUTIONS AND OBSERVATIONS

Caution: Sterility of the dressing is guaranteed unless the pouch is damaged or opened prior to use. DO NOT use if either has occurred. The product is intended for single use only.

Do not use after expiry date.

In the case of an intolerance to the dressing, remove and clean the area carefully.

During the body's natural healing process, non-viable tissue is removed from the wound (autolytic debridement), which could initially make the wound appear larger. If the wound continues to grow larger after the first few dressing changes, consult a healthcare professional. The wound should be inspected during dressing changes. Consult a healthcare professional if there are signs of infection (increased pain, increased redness, wound drainage); excessive bleeding; an unexpected change in wound colour and/ or odour; irritation (increased redness and/or inflammation); sensitivity (allergic reaction); no signs of healing.

If difficulty is experienced on removing the dressing, it should be irrigated or soaked with water or a sterile saline solution.

Because Pro-ionic[®] dressings provide an environment that supports the growth of new blood vessels, the delicate newly-formed tissue may produce blood stained fluid. If excessive bleeding occurs the dressing can be used as a secondary dressing in conjunction with an alginate sheet or a collagen sheet.

Appropriate supportive measures should be taken where necessary (e.g. use of graded compression in the management of venous leg ulcers or pressure relief measures in the management of pressure ulcers, systemic antibiotics and frequent monitoring in the treatment of wound infection, control of blood glucose for diabetic ulcers, etc.).

A change in dressing regimen can result in initially increased levels of exudate which may temporarily require an increased dressing change frequency.

Principelle Matrix Dressing Packaging

Product type	Quantity	Size	Order number
Principelle Matrix	10 dressings	10 x 10cm	PPM1010
Principelle Matrix Border	10 dressings	10 x 10cm (Island 6.8x6.8cm)	PPM1010B

Storage:

Store at room temperature (below 25°C), dry, and keep away from bright light

	For Single Use
	Sterile (Gamma)
	Store in a dry place
	Do not store in sunlight
	Recycled paper packaging

DO NOT use if pouch is damaged or opened prior to use

Principelle Matrix is sterilized by gamma radiation.

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