









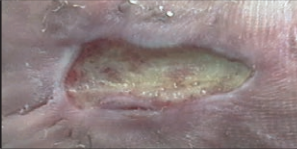


Wound Classification Principelle



	Black wounds	treatment objectives	Removement of black necrosis
	Yellow wounds	treatment objectives	Removement of yellow fibrin layer/pus
	Red wounds	treatment objectives	Acceleration/protection red granulation tissue
	Intact skin	treatment objectives	1. Acceleration/protection pink re-epitheliazation 2. To achieve complete healing/hydration of the skin

(Intact skin)

Wound Classification Principelle

Determine mode of exudate →		Heavily exuding 	Moderately exuding 	Lightly exuding 
Determine the % of colour ↓ ↓ ↓ ↓ of the woundbed		Treatment objectives Absorption No. of dressing changes Daily	Treatment objectives Moist regulation No. of dressing changes 3 - 4 x Weekly	Treatment objectives Rehydration wound bed No. of dressing changes 1 - 2 x Weekly
	Black wound Treatment objectives Removement necrosis	Principelle IF MelMax	Principelle IF MelMax Principelle Matrix	Principelle IF MelMax Principelle Matrix
	Yellow wound Treatment objectives Removement fibrin layer/pus	Principelle IF MelMax Principelle Matrix	Principelle IF MelMax Principelle Matrix	Principelle IF MelMax Principelle Matrix
	Red open wound Treatment objectives Stimulation/protection of granulation tissue	Principelle Matrix	Principelle Matrix	Principelle Matrix
	Pink complete healing Treatment objectives 1. Stimulation/protection of re-epithelialisation 2. To achieve complete healing	NA	NA	Principelle Matrix Pelsan skin Regeneration cream

Product	Pack size	Sizes	Order codes		
MelMax Honey dressing with ionogens formulation	10 dressings 10 dressings 10 dressings	5 x 6 cm 8 x 10 cm 8 x 20 cm	MM050610 MM081010 MM082010	The answer to colonised and infected wounds	Principelle MelMax
Principelle IF Honey based Ionogen Formulation Ointment	10 tubes 10 tubes 10 tubes 10 dressings 10 dressings 10 dressings	10 gram 15 gram 20 gram 5 x 6 cm 8 x 10 cm 8 x 20 cm	PPIF10 PPIF15 PPIF20 PPIF0506 PPIF0810 PPIF0820	Natural Intervention in bacterial load, protease activity and pH	Principelle IF Ointment
Principelle MATRIX Next generation composite wound dressing	10 dressings 10 dressings	10 x10 cm 10 x10 cm island 6.8 x 6.8 cm	PPM1010 PPM1010B	Pro-ionic Wound Care Technology modulation of proteins and ions	Principelle MATRIX
Principelle Pelsan / Pelsan+ / Celan Skin Regeneration Cream	container Pelsan container Pelsan 10 tubes Pelsan+ container Celan	75 ml 150 ml 30 gram 30 ml	PP75 PP150 PPP30	The smart skin regenerator	Principelle Pelsan Principelle Pelsan+ Principelle Celan